

Universal Headrest

Within the disability groups serviced by the dental services there exists a substantial proportion of patients who have spinal or other conditions that lead them to adopt a head position that is not central. These patients may also be the most severely disabled - this combination can lead to difficult treatment scenarios.

The Universal headrest is able to adopt any angle by virtue of its universally jointed headrest pad and take up any lateral position by sliding the headrest along a curved support bar. When adjusted to the correct position it can be locked in place.

The headrest pad has a profiled surface to give full support to both head and neck, this is seen as most important in this compromised group of patients. It is not intended that the Universal Headrest should be for general use but to be fitted as required by removing the conventional headrest.

- Easily fitted to the Compact Wheelchair Recliner
- Makes almost impossible patient cases treatable
- Uses the profiled head and neck support headrest cushion
- The headrest pad is fully articulated
- Full range of lateral motion possible
- Easily operated thumb screws to lock all moving parts in position
- Machinery Directive compliant
- CE and FDA compliant

Installation

- The headrest can replace the supplied standard headrest in the Compact Wheelchair Recliner when required
- Unscrew the locking knob fully and remove it
- Pull the headrest stem out of the locking collars and replace it with the Universal Headrest
- Refit the locking screw



Universal Headrest



Standard Headrest



Attaches to the Compact Wheelchair Recliner with ease to provide comfortable head support

Call us today to arrange a free comprehensive demonstration of how to use our Universal Headrest!